


Restylane

CREATE YOUR

signature
CHIN & JAWLINE

#InspiredByYou

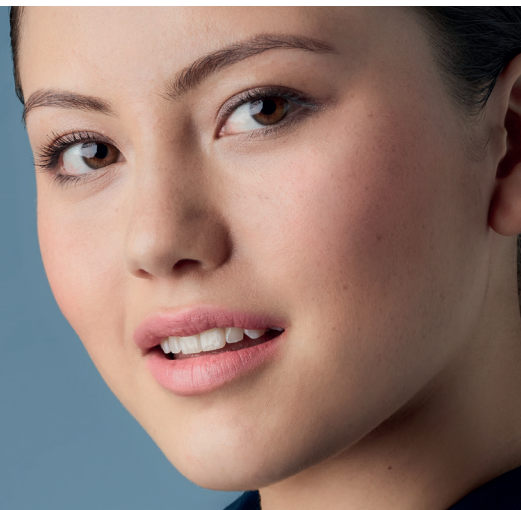

SHAPE. STRUCTURE. DEFINITION.

Dermal fillers in the lower face can bring balance and proportion to the whole face. They can re-shape and define the jaw line, strengthen loose, sagging skin and enhance your profile. With Restylane® Lyft™ and Restylane® Volyme™, your clinician can create a sculpted, lifted appearance that's unique to you.



Before treatment



After treatment



FAQs

1 DO FILLERS HURT?

Modern fillers contain a local anaesthetic called lidocaine for a more comfortable treatment. Our carefully designed needles also minimise pain.

2 HOW LONG WILL MY FILLERS LAST?

Treatment typically lasts up to 12 months but this will depend on the amount and type of Restylane® your practitioner selects.¹ Many people like to have a maintenance treatment before this time to maintain their results.

3 SHOULD I AVOID ANYTHING AFTER TREATMENT?

Until signs of inflammation disappear we recommend you avoid exposing the treated area to extreme heat like sunbathing, saunas or steam baths.

4 ARE THERE ANY SIDE EFFECTS?

The most common side effects are treatment site reactions like swelling, bruising, redness and tenderness. These usually resolve within a week.

5 WHAT'S THE DIFFERENCE BETWEEN PRODUCTS?

Dermal fillers are made of hyaluronic acid, a naturally occurring sugar in the skin that's responsible for hydration. We have the world's most diverse range of dermal fillers^{2,3} which means whatever treatment you're after, you can work with your clinician to create a bespoke treatment tailored specifically for you.

6 HOW IS TREATMENT ADMINISTERED?

Treatment can either be given via a needle or cannula (a flexible, blunt-ended tube). While needles offer more precision, cannulas reduce the likelihood of bruising and are less painful.

If you are concerned about any unexpected complications or side effects, contact your clinician. Adverse events should also be reported to: Galderma (UK) Ltd, Meridian House, 69-71 Clarendon Road, Watford, Hertfordshire, WD17 1DS
Switchboard: +44(0)1923 208950 Email: info.uk@galderma.com.

References

1. Data on file (MA-33947).
2. Data on file (MA-33939).
3. Öhrlund A. Poster presented at AMWC 2019.

Restylane